

#7000-MVP Manual Vacuum Pump

Specifications

Intended Use

- Extraction of hematocele and hydrops in conventional medical rescue
- Extraction of foreign bodies and sputum sample liquid in the trachea
- Extraction of subcutaneous blood/ pus/ venom extraction
- Negative pressure device



Features & Benefits

- Light and portable suction unit that can be easily operated with one-hand, leaving the other hand free for additional urgent tasks
- Easy to operate and adjust suction pressure, using the adjustment knob
- Limit value of negative pressure: $\geq 39.9\text{KPA}$
- Disposable container volume: 300ML

Directions for Use

- Test suction power before use
- Choose the appropriate size catheter and attach to the adapter, and the bottle should be attached to the pump
- Gently pump the sputum and other liquids from patients oral cavity
- When finished pumping, gently remove the suction tube, and discard catheter and bottle into plastic bags