

REVOLUTIONARY



Needle-Free Injector ... for Patient Comfort

*Clinically proven to reduce the fear and trauma associated
with needle & syringe injections*

... What is the MadaJet?

- A handheld device for injecting medication without a needle or syringe
- Creates a fine jet stream that penetrates the skin & is virtually PAINLESS




- For use with anesthetics, steroids, & various other medicaments
- Produces higher levels of tissue anesthesia using lesser volumes
- Minimizes tissue trauma around the injection site
- Can be sterilized by autoclaving
- 1-Year Warranty



Physiological Advantages:

The MadaJet is not a Topical injection device. Each injection delivers a volume of **0.1 cc**, at a depth of **4.5 - 5 mm** below the epithelium. At the base of each infiltration, a wheal is formed that measures approximately **5 - 6 mm** in diameter. Thus, each injection into the tissue forms an inverted cone as shown in the diagram to the right.

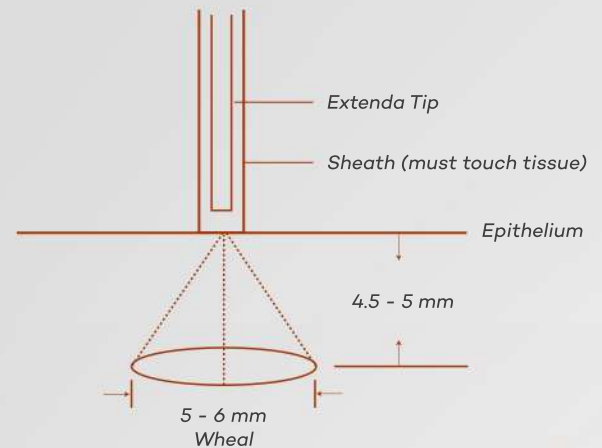

Product # 401UR Includes:

- (a) Assembled MadaJet
- (b) Custom carrying case
- (c) Stand
- (d) Stylet
- (e) Wrench
- (f) Extra 3/4" Extenda Tip® with stainless steel sheath
- (g) Two extra Pyrex fill-chambers
- (h) MadaCide-FD disinfectant/cleaner
- (+) Set of finger cots (NOT DISPLAYED)

Supporting Materials:

- New technique to deliver rapid onset of profound local anesthesia to the patient undergoing vasectomy
- Decreases the fear of pain and enhances the popularity of vasectomy
- High patient satisfaction, as reflected in low pain scores
- Unique stainless steel tip that fits over the vas deferens


NO-NEEDLE JET ANESTHETIC TECHNIQUE FOR NO-SCALPEL VASECTOMY

Ronald S. Weiss, M.D.
Philip S. Li, M.D.

NO-NEEDLE ANAESTHESIA FOR NO-SCALPEL VASECTOMY:

An Instructive Guide for Surgeons

Ronald S. Weiss, M.D.
Philip S. Li, M.D.

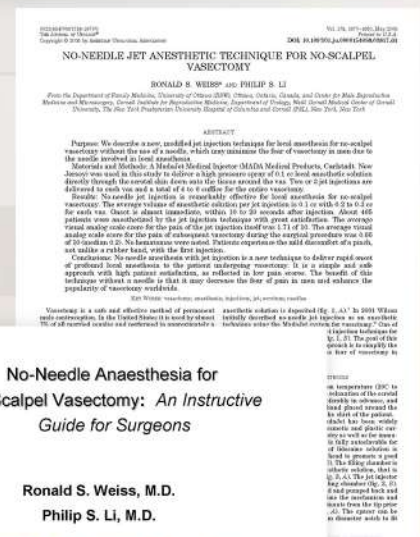

Copyright © 2011

The Weiss Clinic, Ottawa, Canada