


SITES FOR INJECTIONS OF THE DENTAL MADAJET XL

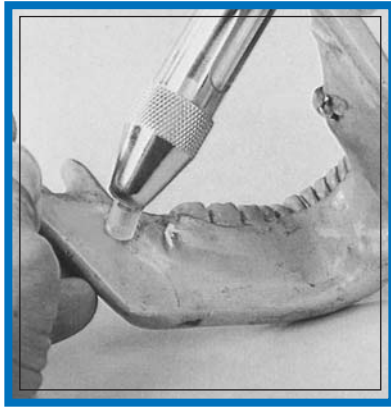


MANDIBULAR FORAMEN

(Two Injections)

#1. Inject anteriorly to the foramen where you would first start your needle insertion.


#2. Inject in the close proximity of the foramen.

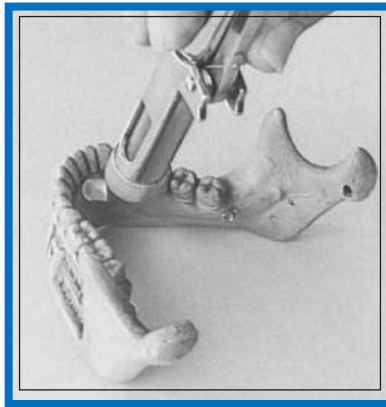

INFERIOR MAXILLARY

(Lingual Surface)

Inject at the inter-apical (inter radicular) area towards bony underlying structures.

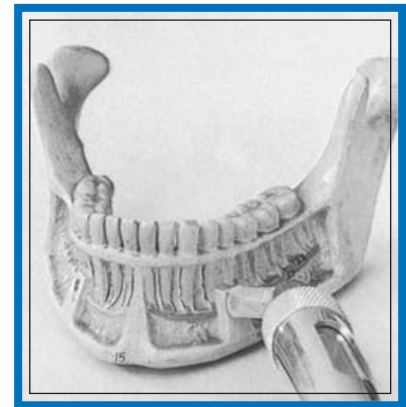


INFERIOR MAXILLARY

(Mental Foramen)

Inject directly over the foramen area for quadrant anesthesia.


For even deeper anesthesia follow with MadaJet Lingual injection.

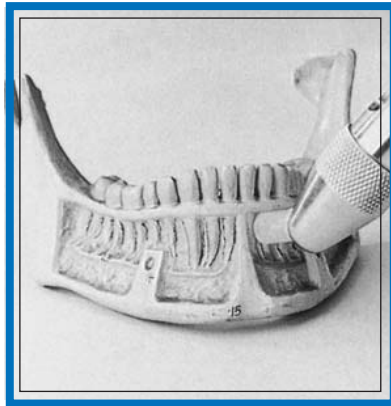

INFERIOR MAXILLARY

(Incisors)

Injection at the inter-apical (inter-radicular) area and follow with labial injection at the same anatomical position for more profound anesthesia, if required.

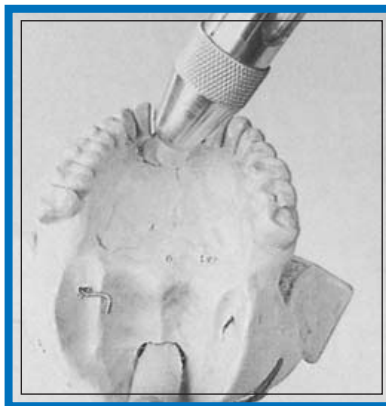


SUPERIOR MAXILLARY

(Nasal Palatine Foramen)

Make two injections with the MadaJet XL, one on each side of the papillae.

Follow with 2-3 injections in line where your needle & syringe is to be directed.

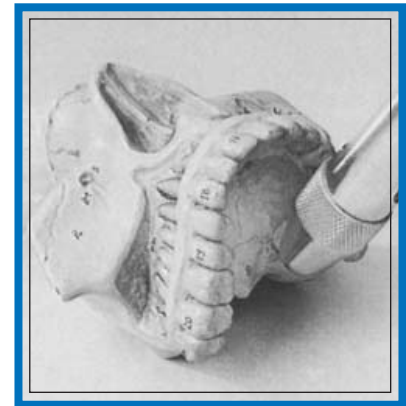


SUPERIOR MAXILLARY

(Posterior Nasal Foramen)

Make two injections with the MadaJet XL, one on each side of the papillae.

Follow with 2-3 injections in line where your needle & syringe is to be directed.

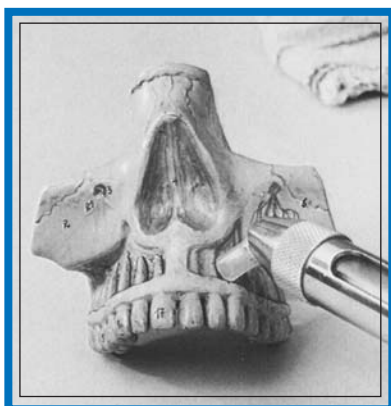

SUPERIOR MAXILLARY

(Ant. Palatine Foramen)

Inject anteriorly and posteriorly to foramen site or directly over foramen areas.



PULP INFILTRATION

When the pulp areas in root canal procedure is still sensitive, remove sheath and inject directly into canal, repeat if needed.

For further techniques and information please request:

MadaJet XL Dental Presentation and Dental Applications and Techniques,
at No Charge.



MADA, INC.

DOMESTIC/INTERNATIONAL

625 Washington Avenue, Carlstadt, NJ 07072

Tel: 201-460-0454 Fax: 201-460-3509

Toll Free: 800-526-6370